



IVTAG Device Instructions for Use


Application

1. Remove IVTAG from storage container.
2. Use ball point pen or fine point permanent marker to write information on IVTAG designated areas...let ink dry.
3. Wrap extended tab around desired tubing and press snaps together, securing IVTAG to tubing.
4. IVTAGs are usable for all types of Vascular Access Device tubing. Use on appropriate device.

- Non-Sterile Product unless otherwise indicated in alternate packaging.

- Keep out of reach of children. In case of accidental swallowing or choking, seek professional assistance.

Made in the USA.

IVTAGS, LLC 

www.IVTAGS.com

Model No: IVTAG1-325916, IVTAG1-325916B,

IVTAGS2-325916, IVTAGS3-32534

IVTAGS and IVTAGS.com are registered trademarks of IVTAGS, LLC.



Do not reuse



No Latex



Do not use if
package is damaged